



Serafol® ABO and ABO+D Bedside cards for identity proof

Identity proof

Bedside cards

Blood transfusion

Higher patient safety during blood transfusions

- **Clearly visible results of blood typing within a few seconds**

Strong and rapid agglutination: Clearly visible result is readable immediately!
No loss of time caused by unnecessary waiting.

- **No further lab equipment necessary**

- **Saving of energy costs**

No need for refrigeration:
Storage is at room temperature!

- **Easy to use**

No canula required for puncture of a membrane:
There is no risk of physical injury!

- **Environmentally friendly paper-based bedside card**

No plastic waste





- **Two individual determinations on 1 card possible**

2 tests on 1 twin card: e.g. for blood bag/donor and patient/recipient
or for two recipients, since the double card can be separated in the middle.

- **Can be archived for documentation purposes**



Easy workflow at a glance

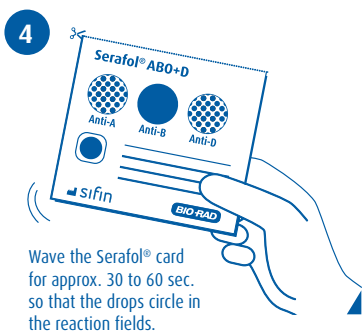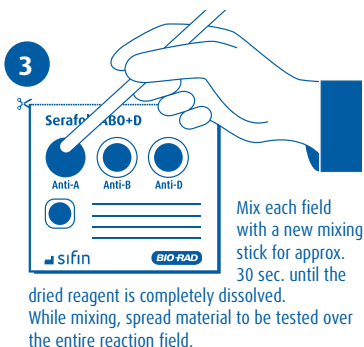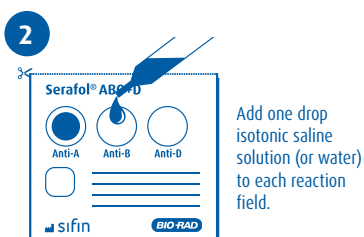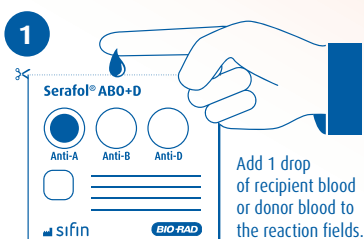

Serafol® AB0 and AB0+D for identity proof

In many countries, regulations governing the use of blood products require identity checks immediately prior to blood transfusion. The Serafol® hemagglutination test is used for the confirmation of earlier ABO and D blood grouping^{*)} of the recipient and ensures the compatibility of blood types of the recipient and the blood to be transfused.

Maximum quality with Serafol® AB0 and AB0+D

✓ Serafol® is easy and ready-to-use

Capillary-, vein-, or donation blood is applied directly to the reaction fields.

✓ Reliable results due to auto-control field


False positive results caused by cold-reacting autoantibodies or the phenomenon of rouleaux formation can be clarified immediately at the bedside due to the Serafol® auto control field.

✓ Sensitivity and Reliability

No false positive results!
Does not detect „acquired B“!
Immediate detection of A₁ and A₂ (A₃ or A_x weak positive or negative)!
Detection of non-specific reactions by plasma proteins.

✓ Archivable

For documentation, the reaction mixes can be dried and covered by a self-adhesive, see-through foil cover.

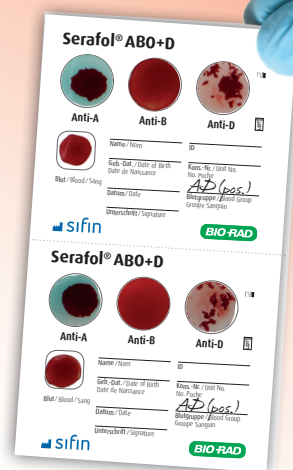

Serafol® product line

Product	Description	Art. No.	Packing
Serafol® AB0 sticks and foils included	For the ABO system. 1 kit contains: 1 twin card, 2 cover foils and 2 mixing sticks. 10 double or 20 single determinations.	BG1721	10 x 1 kit
Serafol® AB0	For the ABO system. 50 double or 100 single determinations.	BG1722	50 cards
Serafol® AB0+D	For the ABO+Rh system. 50 double or 100 single determinations.	BG1723	50 cards
Mixing sticks	Used to stir each bedside card's reaction field until the reagents are completely dissolved.	BG1712	100 sticks
Self-adhesive foil cover	For documentation, the bedside card can be covered by a self-adhesive foil cover.	BG1713	100 foils

^{*)} Please find our test reagents for blood grouping on www.sifin.de.

Manufacturer / Distributor

sifin diagnostics GmbH

Distributor

Bio-Rad Medical Diagnostics GmbH